

# Flexi-Q eMU-P

## Electromechanical Multi-use Auto-injector for drugs in pre-filled syringes

Smart features, wireless connectivity

Technical Data	
Device description	Electromechanical multi-use Auto-injector for drugs in pre-filled syringes (PFS)
Weight (excl PFS/drug)	158g
Overall dimensions	120mm (length) x 60mm (width) x 20mm (thickness)
Grip area	50mm (width) x 35mm (thickness)
Primary container	1mL "long" PFS (pre-loaded into single-use Cassette)
Route of administration	Subcutaneous
Power source	Rechargeable battery
Service life	30 injections per battery charge



## User Handling Steps

1



Insert Cassette into Cassette port

2



Remove Cassette Cover

3



Push Device against injection site and press Inject button

4



Eject Cassette and dispose of in appropriate container

## Key Features



Fully automatic  
injection process



Advanced  
user interface



Smart device  
with wireless  
connectivity

### Injection Parameters

Dosing options	Fixed dosing or user-selectable dosing
Injection duration	3 sec – 15 sec (fixed or user-selectable)
Viscosity	Low to high viscosity"
Injection volume range	0.1mL – 1.0mL
Injection volume increment	0.05mL

### Needle

Exposed needle length	Up to 12mm
Needle shield	Rigid needle shield (RNS) or soft needle shield (SNS)
Needle safety	Locked Needle Shield (passive safety) following completion of injection
Activation method	Injection button
User guidance	On-screen step-by-step instructions (text and icon)
User feedback	Visual (on-screen) and audible indicators
Number of user handling steps	4 handling steps, including disposal of used PFS
Device customization options	External geometry and colors